Sinclair Dental Mirror Heads and Handles Care and Sterilization Instructions

1 General Instrument Care

Sinclair Dental mirror heads and handles are manufactured from surgical grade stainless steel which offers superior durability and corrosion resistance, but they can be damaged if exposed to certain corrosive chemicals or conditions.

The following chemicals should **not** be used to clean or disinfect our mirror heads and handles:

Sodium Hypochlorite (bleach)	Tartaric Acid (stain/tartar remover)
Chemical Chlorides	Citric Acid
Iodine	Lysol
Mercury Salts	Phenol
Potassium Permanganate	Potassium Thiocyanate
Aqua Regia	Sulfuric Acid
Hydrochloric Acid	

- Mirror heads and handles should be handled with care and thoroughly inspected for damage if dropped.

2 Cleaning and Disinfection

All mirror heads and handles should be thoroughly cleaned after each use and before sterilization using one of the methods listed below:

- Ultrasonic Cleaning
- Automated Washing Equipment
- Manual scrubbing in warm soapy water
- Corrosive cleaners and detergents should not be used due to risk of chemical reactions with the stainless steel.
- Abrasive bristle (wire) brushes are not recommended as the surface finish of the mirror heads and handles could be damaged.
- All mirror heads and handles must be thoroughly dried before sterilization.

3 Sterilization

- All mirror heads and handles should be sterilized in a commercial sterilization pouch or instrument cassette according to the following parameters:
 - Steam Autoclave cycle of 20 minutes @ 121°C or 5 minutes @ 134°C.
- Mirror heads and handles should never be sterilized along with non-stainless steel metal items.

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